# VICTO

Adjustable Artificial Sphincter

## ADJUSTMENT KIT

INFORMATION FOR SURGEONS - INSTRUCTIONS FOR USE  
**ENGLISH** ■

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VICTO

Adjustable Artificial Sphincter

VICTO and VICTO+ are adjustable artificial urinary sphincters (AUS) used to treat male stress urinary incontinence caused by intrinsic sphincter deficiency (ISD) in cases such as post-prostatectomy incontinence.

## ADJUSTMENT KIT (KIT-AD-06)

This accessory kit contains the necessary materials to adjust the system internal pressure by adding or extracting sterile saline solution. It includes the following sterile and disposable items:

- 1 (one) 6 ml Syringe
- 2 (two) non-coring injection needles: 25 G x 5/8"

VICTO/VICTO+ is pressurized by injecting sterile saline solution through the self-sealing port in the base of the control pump. The pressure in the device depends upon the amount of fluid injected. This feature allows the device to be set at any desired pressure without having to change the PRB.

For more information about the use of VICTO accessories, please refer to the instructions for use supplied with VICTO/VICTO+ Starter Kits.

### EXPLANATION OF SYMBOLS USED

	CATALOG NUMBER		DO NOT RESTERILIZE
	BATCH-LOT-ID NUMBER		MANUFACTURER
	CAUTION		SERIAL NUMBER
	EXPIRY DATE - USE BY		CONSULT INSTRUCTIONS FOR USE
	SINGLE USE		DO NOT USE IF PACKAGE IS DAMAGED
	MANUFACTURE DATE		KEEP AWAY FROM SUNLIGHT
	STERILE. STERILIZATION METHOD: ETHYLENE OXIDE		KEEP DRY

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**Sphinx Medical Limited,**